Gravurerkennungsgerät

Engraving Recognition Unit

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Intended use

- This unit serves the recognition of lens engravings. It should not be used for other purposes.

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General safety

- Please read the safety instructions and the operating instructions carefully before connecting the unit to the mains.
- Keep these operating instructions in a safe place.
- If this unit is relocated, these instructions should also be handed over.
- The packaging material and the unit itself are not toys. Keep plastic bags away from children. There is a risk of suffocation. Children may not play with batteries or LED's. Swallowing such parts is particularly dangerous.
- Never try to repair the unit yourself. Only qualified personnel should carry out technical repairs.

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Environmental conditions and installation conditions

- The unit must be set up on a solid base with no danger of falling over.

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Precautions with respect to electrical, magnetic and optical risks

- The unit works with commercial 1.5 V AAA batteries. Alternatively, a mains adapter (7 V, 0.5 A) can be used. Please always read the operating instructions of the mains adapter in this case.
- Avoid looking directly at the LED.

The generally applicable, local accident prevention regulations and guidelines must also be followed in addition to these instructions.

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Included in the delivery

- **VISUALIZE** Engraving Recognition Unit
- 4 batteries 1,5 V, AAA
- These operating instructions

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Description of the unit

The Visualize Engraving Recognition Unit is the ideal unit to make invisible engravings on lenses* visible again. It is small and handy and suitable for use both in the sales area and in the workshop ; it can be used for either lenses or spectacles. The unit is operated with standard batteries but can also be supplied with power using an optional power pack. The unit switches automatically off after about 5 minutes.

* The unit is designed for the recognition of engravings in plastic lenses. It can be used with restrictions for the recognition of engravings in lenses from mineral glass.

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Function

Place the lens on the lens support and align it (Fig. 1). The lens can be held against the lens support either from above or below. It is therefore possible to recognise engravings ideally which are applied either to the front or the rear of the lenses. It is thus also possible to mark the lens from either side (Fig. 2).

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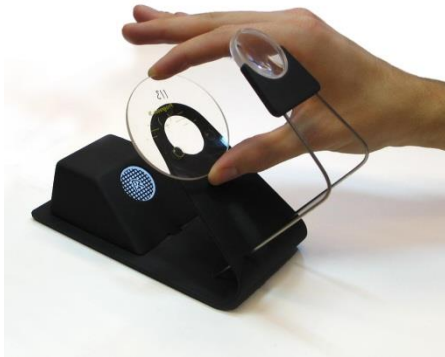
Checking lens

Place the lens on the lens support – can be held against the lens support either from above or below. It is therefore possible to recognise engravings ideally which are applied either to the front or the rear of the lenses

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Marking lens

Possible to mark the lens from either side

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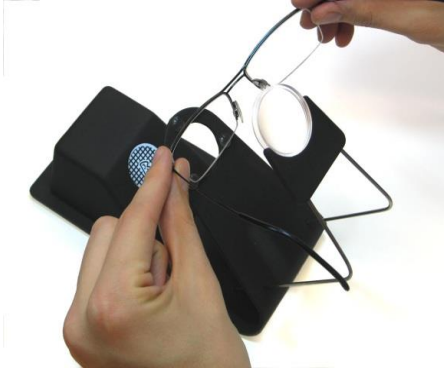
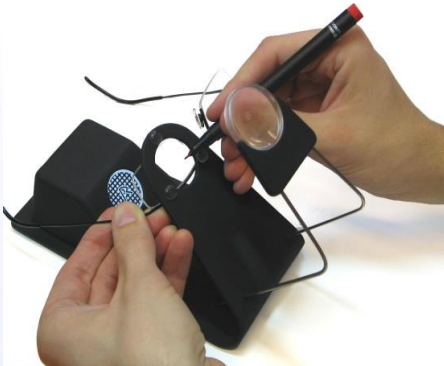
Changing the batteries

Open the cover by pressing the rear side and simultaneously slightly raising it from the unit; the batteries are now accessible.

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Accessories

A mains adapter (power pack) can be used as a useful accessory 7V 0,5A

It can be purchased locally from an electrical store.

Cleaning

Do not use any aggressive or scouring cleaning agents to clean the unit.

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